

# YOUR ENDO TREATMENT USING THE SICAT ACCESSGUIDE

Please note the following workflow to ensure a positive result of your treatment:

1

## PLACEMENT OF ACCESSGUIDE

Guarantee a correct and tight fit throughout the procedure.



2

## COLOUR MARKING

Mark and verify the entry point of your treatment.



3

## REMOVE ENAMEL

Remove the enamel or the restoration at the marked area.



4

## TRANSFER DRILL DEPTH

Use a ruler and a rubber stop.



5

## INSERT WITHOUT ROTATION


Do not start the drill until it is in contact with the tooth.



6

## WORK INTERMITTENTLY

Regularly check for correct drilling progress.

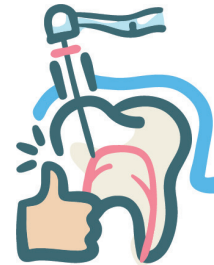

SICAT 

**MANUFACTURER**

SICAT GMBH & CO. KG  
FRIESDORFER STR. 131-135  
53175 BONN  
GERMANY  
WWW.SICAT.COM

**LOCAL SUPPORT**  
SGL@SICAT.COM

ART-NR: 1067011EN  
DOC-ID: DD33POG002  
REVISION: 2021-02-08

## YOUR ENDO TREATMENT USING THE SICAT ACCESSGUIDE

Please also observe the following instructions during treatment:



▶ **OCCUSAL HEIGHT**

Make sure that there is **enough space** available.

▶ **DO NOT LEVER**

Always guide the drill **straight** through the drill sleeve without changing the sleeves position.

▶ **WATER COOLING**

Perform the drilling under **constant cooling**.

▶ **REVOLUTIONS PER MINUTE**

Follow the manufacturer's instructions. SICAT recommends a **maximum of 5000 RPM**.

▶ **FLUSH THE DRILL PATH**

**Regularly** remove any resulting debris.



**FOR COMPLETE PRODUCT INFORMATION, PLEASE USE  
THE SICAT ACCESSGUIDE INSTRUCTIONS FOR USE.**

SICAT 

 **MANUFACTURER**

SICAT GMBH & CO. KG  
FRIESDORFER STR. 131-135  
53175 BONN  
GERMANY  
WWW.SICAT.COM

**LOCAL SUPPORT**

SGL@SICAT.COM

ART-NR: 1067011EN  
DOC-ID: DD33POG002  
REVISION: 2021-02-08