

YOUR ENDO TREATMENT USING THE SICAT ACCESSGUIDE

Please note the following workflow to ensure a positive result of your treatment:

1

PLACEMENT OF ACCESSGUIDE

Guarantee a correct and tight fit throughout the procedure.



2

COLOUR MARKING

Mark and verify the entry point of your treatment.



3

REMOVE ENAMEL

Remove the enamel or the restoration at the marked area.



4

TRANSFER DRILL DEPTH

Use a ruler and a rubber stop.



5

INSERT WITHOUT ROTATION


Do not start the drill until it is in contact with the tooth.



6

WORK INTERMITTENTLY

Regularly check for correct drilling progress.

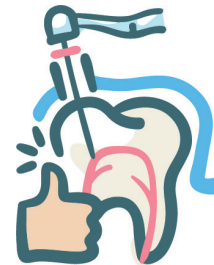

SICAT 

MANUFACTURER
SICAT GMBH & CO. KG
BRUNNENALLEE 6
53177 BONN
GERMANY
WWW.SICAT.COM

LOCAL SUPPORT
SGL@SICAT.COM

ART-NR: 1067011EN
DOC-ID: DD33POG002
REVISION: 2019-12-04

YOUR ENDO TREATMENT USING THE SICAT ACCESSGUIDE

Please also observe the following instructions during treatment:



▶ **OCCUSAL HEIGHT**

Make sure that there is **enough space** available.

▶ **DO NOT LEVER**

Always guide the drill **straight** through the drill sleeve without changing the sleeves position.

▶ **WATER COOLING**

Perform the drilling under **constant cooling**.

▶ **REVOLUTIONS PER MINUTE**

Follow the manufacturer's instructions. SICAT recommends a **maximum of 5000 RPM**.

▶ **FLUSH THE DRILL PATH**

Regularly remove any resulting debris.



**FOR COMPLETE PRODUCT INFORMATION, PLEASE USE
THE SICAT ACCESSGUIDE INSTRUCTIONS FOR USE.**

SICAT 



MANUFACTURER
SICAT GMBH & CO. KG
BRUNNENALLEE 6
53177 BONN
GERMANY
WWW.SICAT.COM

LOCAL SUPPORT
SGL@SICAT.COM

ART-NR: 1067011EN
DOC-ID: DD33POG002
REVISION: 2019-12-04