

# SICATJMT<sup>+</sup>




## ***SICAT JMT<sup>+</sup> - PREPARATION OF THE SICAT FUSION BITE***

QUICK GUIDE | English

# PREPARATION OF THE SICAT FUSION BITE

## QUICK GUIDE

Both during the CBCT scan and prior to recording the patient-specific mandibular movements using SICAT JMT<sup>+</sup>, the patient must wear a SICAT Fusion Bite to register the 3D X-ray scan with the mandibular movement data. The SICAT Fusion Bite must be prepared as follows:

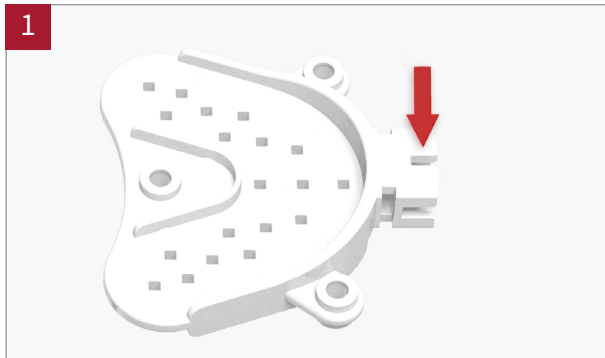


- Always position the SICAT Fusion Bite **with the groove facing upwards**. Otherwise, it will not be possible to connect it to a SICAT Fusion Bite Adapter later on.



- Apply bite registration material onto the **upper surface** of the SICAT Fusion Bite.



- Take the bite impression of the patient's maxilla.

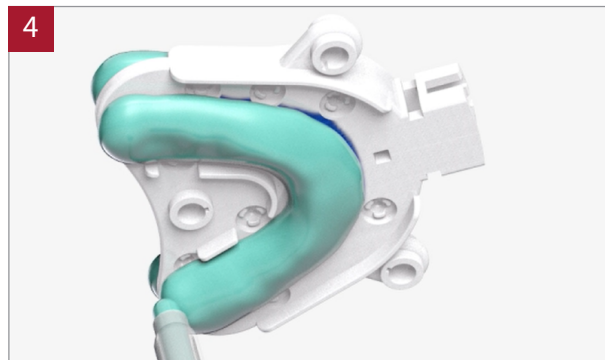

- Apply bite registration material onto the **lower surface** of the SICAT Fusion Bite.



- Position the SICAT Fusion Bite on the patient's maxilla and **take a bite registration of the mandible** by asking the patient to bite on the material.



- Remove excess material.

SICAT 

 MANUFACTURER  
SICAT GMBH & CO. KG  
FRIESDORFER STR. 131-135  
53175 BONN, GERMANY  
WWW.SICAT.COM

CE 0197

SUPPORT  
TELEPHONE: +49 228 286206600  
FAX: +49 228 286206971  
E-MAIL: SUPPORT@SICAT.COM



DH71QUG006  
REVISION: 2021-02-16  
PAGE: 2 / 2