

# SICATJMT<sup>+</sup>




## **SICAT JMT<sup>+</sup> - PREPARATION OF THE T-ATTACHMENT**

QUICK GUIDE | English

# PREPARATION OF THE T-ATTACHMENT

## QUICK GUIDE

In order to connect the lower jaw sensor for recording patient-specific jaw movements, the T-attachment must be fixed to the vestibular surface of the patient's lower teeth:

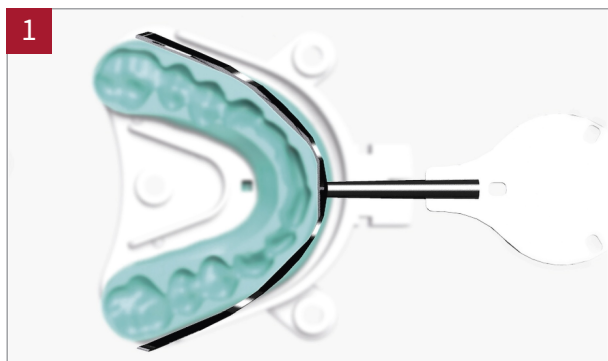

1

- Adjust the T-attachment to the patient's mandibular arch.



2

- Check the shape of the T-attachment in the patient's mouth.



3

- Place the SICAT Fusion Bite in the patient's mouth to determine the height for the connection of the T-attachment. **There must be no interferences between SICAT Fusion Bite and T-attachment.**
- Dry the vestibular tooth surface of the lower teeth.



4

- Apply flowable composite to the T-attachment.



5

- Connect the T-attachment to the vestibular surfaces of the lower teeth.



6

- Remove the SICAT Fusion Bite from the patient's mouth.
- Check that the T-attachment is securely fixed.

SICAT 

 MANUFACTURER  
SICAT GMBH & CO. KG  
FRIESDORFER STR. 131-135  
53175 BONN, GERMANY  
WWW.SICAT.COM

CE 0197

SUPPORT

TELEPHONE: +49 228 286206600  
FAX: +49 228 286206971  
E-MAIL: SUPPORT@SICAT.COM



DH71QUG005  
REVISION: 2021-02-16  
PAGE: 2 / 2