



UPDATING THE IMPLANT DATABASE

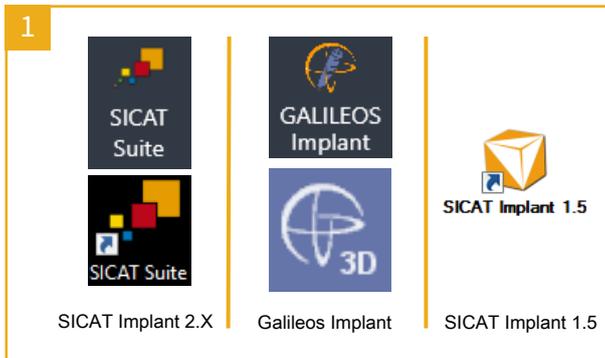


Find out what software you are using:
SICAT Implant 2.X can be recognized by the icons on the left-hand side for SICAT Suite in Sidexis 4 or as a separate application.
Galileos Implant can be recognized by the icons in the center for Galileos Implant in Sidexis 4 or in SIDEXIS XG.
SICAT Implant 1.5 can be recognized by the symbol on the right-hand side.

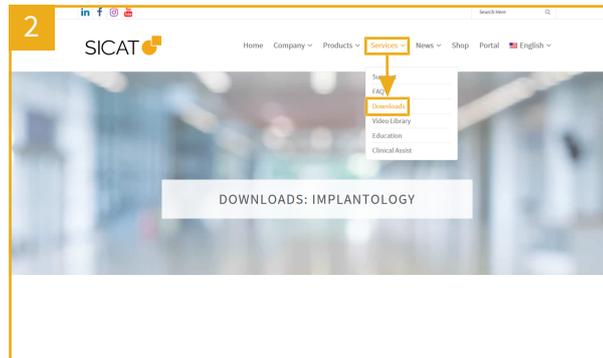


On the SICAT homepage <https://www.sicat.com> under **Services**, click on **Downloads**.
 Click on **IMPLANTOLOGY**.

SICAT Implant 2.X:

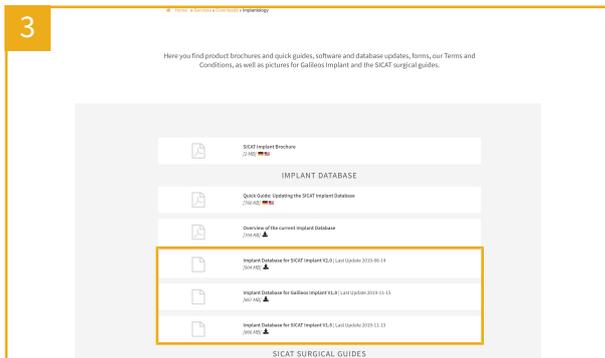


Scroll down to the **IMPLANT DATABASE** area. Click on the download icon for the entry that corresponds to your software and save the file. Double-click on the saved file.

SICAT Implant 2.X:

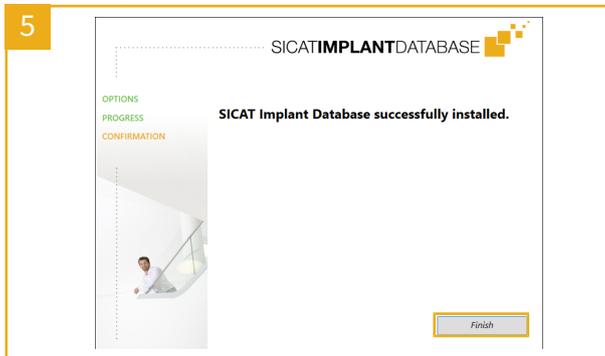


Wait until the installation program closes the window containing the progress bar and click on **Finish** in the last window.

Galileos Implant and SICAT Implant 1.5:

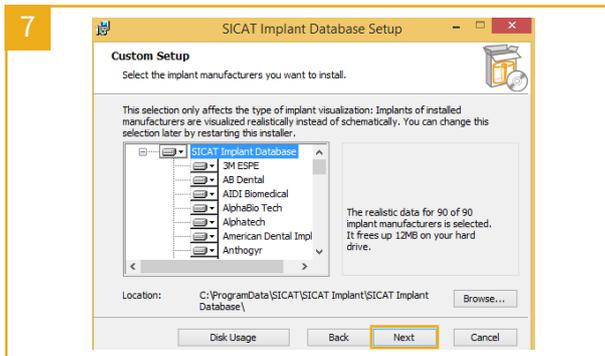

Read and accept the license agreement by selecting the appropriate check box and clicking on **Next**. Click on **Next** again and click on **Install** in the next window.

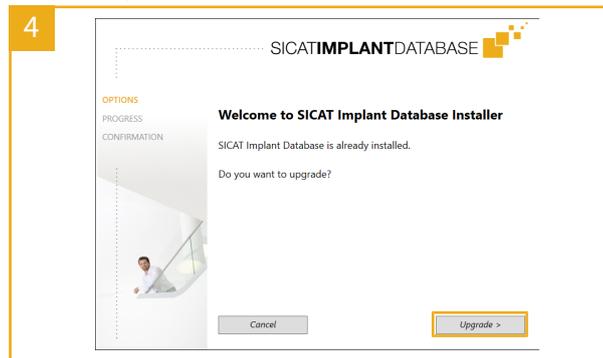

Steps 4 to 5 apply to SICAT Implant 2.X only. If you are using Galileos Implant or SICAT Implant 1.5, follow steps 6 to 8.
 In the Welcome window, click on **Upgrade**.

Galileos Implant and SICAT Implant 1.5:



In the Welcome window, click on **Next**.

Galileos Implant and SICAT Implant 1.5:



Wait until the installation program closes the window containing the progress bar and click on **Finish** in the last window.

SICAT

MANUFACTURER
 SICAT GMBH & CO. KG
 FRIESDORFER STR. 131-135
 53175 BONN, GERMANY
 WWW.SICAT.COM

CE 0197

SUPPORT
 WWW.SICAT.COM/SUITESUPPORT



DX2XQUG002
 REVISION: 2021-02-26
 PAGE: 1 / 1