


Ordering SICAT OPTISLEEP based on surface data

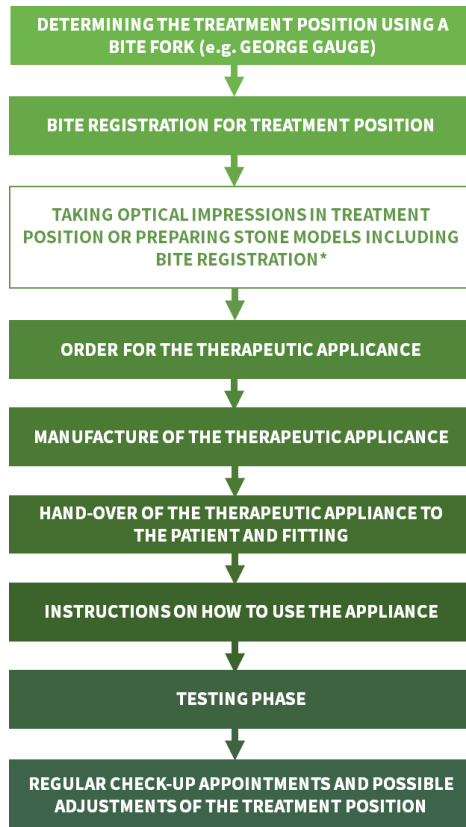

*Taking digital intraoral scans

Use your intraoral scanner to take a **scan of the complete mandible**, a **scan of the complete maxilla** and a **scan of the buccal bite with bite fork** (treatment position) on both sides if possible.

Note: The registration material on the bite fork should **not cover** the buccal tooth surfaces so that the scanner software can correctly correlate the bite. **Remove any interfering or excess material if necessary.**



Manufacturer
SICAT GmbH & Co. KG
Friesdorfer Str. 131-135
53175 Bonn, Germany
www.sicat.com

Document ID: Addendum-EN-
(DD32IFU002-2022-03-28)-2023-04-20
Refers to instructions for use:
D32IFU002-2022-03-28

SICAT 